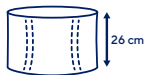
THUASNE

LombaSkin

fr	Ceinture de soutien lombaire - port à même la peau	5
en	Next-to-skin lumbar support	6
de	LWS-Stabilisierungsothese - für direktes Tragen auf der Haut.....	7
nl	Compacte rugbrace voor de onderrug, om op de blote huid gedragen te worden	8
it	Cintura di sostegno lombare - indossabile a contatto con la pelle.....	9
es	Faja de sujeción lumbar - uso sobre la piel.....	10
pt	Cinta de suporte lombar - uso em contacto directo com a pele.....	11
da	Lændestøttebælte - bæres direkte mod huden	12
fi	Lannerangan tukivyö - pidetään iholla	13
sv	Ryggördel - bärs närmast huden	14
el	Ζώνη οσφυϊκής υποστήριξης - εφαρμογή σε επαφή με το δέρμα	15
cs	Bederní podpůrný pás - aplikovatelný přímo na tělo	16
pl	Orteza lędźwiowa - do noszenia bezpośrednio na skórze	17
lv	Muguras jostas daļas atbalsta josta - valkāt tieši uz ādas.....	18
lt	Palaikomas juosmens diržas - dėvimas tiesiai ant odos.....	19
et	Nimmepiirkonna tugikorsett - kanda vastu nahka	20
sl	Oporni križni pas - namestitev na kožo	21
sk	Podporný drienkový pás - možnosť nosenia priamo na pokožke	22
hu	Ágyéki fűző - a bőrön is viselhető	23
bg	Лумбален колан - да се носи директно върху кожата	24
ro	Centură de susținere lombară - se poartă direct pe piele	25
ru	Опорный поясничный корсет — надевать изделие непосредственно на тело	26
hr	Lumbalni potporni pojas - nošenje uz kožu	28
zh	腰部支撐帶 - 緊貼皮膚佩戴	29
ar	حزام لدعم أسفل الظهر - للارتداء على البشرة	30

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	1	60 - 80 cm
	2	75 - 95 cm
	3	90 - 110 cm
	4	105 - 125 cm
	5	120 - 140 cm

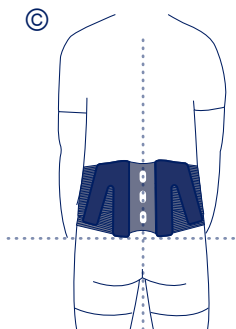
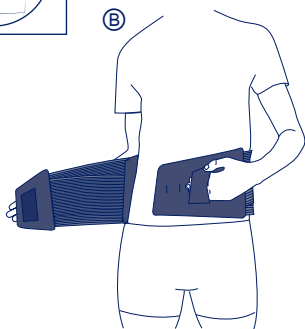
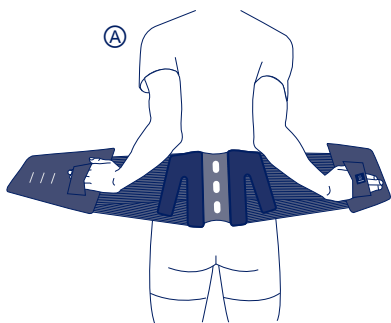
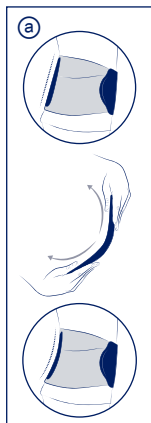
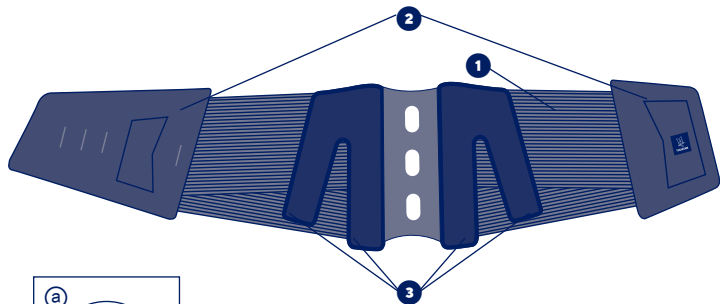
fr	Tour de taille	Hauteurs : 21 cm - 26 cm
en	Waist circumference	Heights: 21 cm - 26 cm
de	Taillenumfang	Höhen: 21 cm - 26 cm
nl	Tailleomvang	Hoogtes: 21 cm - 26 cm
it	Circonferenza vita	Altezze: 21 cm - 26 cm
es	Contorno de cintura	Alturas: 21 cm - 26 cm
pt	Perímetro de cintura	Alturas: 21cm - 26cm
da	Taljemål	Højde: 21cm - 26cm
fi	Vyötärönympärys	Korkeudet: 21 cm - 26 cm
sv	Midjemått	Höjd: 21cm - 26cm
el	Περίμετρος μέσης	Ύψη: 21 εκ. - 26 εκ.
cs	Obvod pasu	Výšky: 21 cm - 26 cm
pl	Obwód w talii	Wysokości: 21 cm - 26 cm
lv	Vidukļa apkārtmērs	Augstumi: 21 cm - 26 cm
lt	Juostmens apimtis	Aukštis: 21 cm - 26 cm
et	Vööümbmõõt	Kõrgused: 21 cm - 26 cm
sl	Obseg pasu	Višina: 21 cm - 26 cm
sk	Obvod pása	Výšky: 21 cm - 26 cm
hu	Derékbőség	Magasság: 21 cm - 26 cm
bg	Обиколка на талията	Височини: 21 cm - 26 cm
ro	Circumferința taliei	Înălțimi: 21 cm - 26 cm
ru	Обхват талии	Высота: 21 см - 26 см
hr	Opseg struka	Visine: 21 cm - 26 cm
zh	腰围	高度 : 21 cm -26 cm
ar	محيط الخصر	الارتفاعات: 21 سم - 26 سم

fr	Compression	Maintien	Education posturale
en	Compression	Support	Postural education
de	Kompression	Halt	Erlernen der Körperhaltung
nl	Compressie	Fixatie	Voorlichting over houding
it	Compressione	Sostegno	Educazione posturale
es	Compresión	Sujeción	Educación postural
pt	Compressão	Manutenção	Educação postural
da	Kompression	Støtte	Postural genoptræning
fi	Kompressio	Tuki	Asentokuntoutus
sv	Kompression	Stöd	Postural träning
el	Συμπίεση	Συγκράτηση	Σωστή στάση σώματος
cs	Komprese	Stabilizace	Správné držení těla
pl	Kompresja	Podtrzymywanie	Edukacja dotycząca postawy
lv	Kompresija	Atbalsts	Stājas korekcija
lt	Kompresija	Palaikymas	Laikysenos taisymas
et	Kompressioon	Hooldus	Rühitreening
sl	Kompresija	Orijem	Učenje telesne drže
sk	Kompresia	Podpora	Cvičenie posturálnych svalov
hu	Szorítás	Támasz	Tartásjavító oktatás
bg	Компресия	Поддръжка	Приучаване към правилна стойка
ro	Compresie	Suținere	Educație posturală
ru	Компрессия	Стабилизация	Исправление осанки
hr	Kompresija	Potpora	Učenje o držanju
zh	压紧	稳定	姿势联系
ar	الضغط	الدعم	تدريب وضعي

fr

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NEXT-TO-SKIN LUMBAR SUPPORT

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Aerated and breathable Combitex fabric. ❶

Abdominal panels with self-fastening surface and finger-loops. ❷

Comfortable dorsal stays. ❸

Composition

Textile components: polyamide - polyester - elastane - polyurethane.

Non-textile components: brushed tempered steel - polyoxymethylene - polypropylene - polyethylene.

Properties/Mode of action

The device relieves low back pain by supporting the spine through an increased intra-abdominal pressure which eases the load on the intervertebral discs.

Indications

Non-specific low back pain (acute, subacute and chronic phases). Prevention of recurrent low back pain in combination with return to activity and/or therapeutic information.

Spinal stenosis.

Spine stability.

Resumption of activity.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

*Do not use on pregnant women.

Do not use in the event of a hiatus hernia.

Do not use in the event of bone cancer with spinal metastases.

Do not use in the event of circulatory, pulmonary or cardiovascular problems in patients for whom an increase in arterial blood pressure is not recommended.

Precautions

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

Do not wear the product while sleeping.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

Measure the waist circumference of the patient at the mi-distance between the iliac crests and floating ribs.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not wear the product in a medical imaging machine.

Undesirable side-effect

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended that a healthcare professional adjust the stays to the shape of the patient's back before the belt is worn for the first time. ❹

Position the belt around the waist, using the finger loops located on the front panels. ❺

The Thuasne logo indicates which way up the belt should be put on.

Pull the finger loops on the abdominal panels simultaneously and evenly.

Fasten the belt, right side over left side, adjusting to the desired compression. ❻

The belt must be centred on the spine: the dorsal stays should be positioned on either side of the spine. ❼

The bottom of the belt must stop at the top of the crease between the buttocks. ❽

The level of compression can be adjusted as required throughout the day.

Care/Maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme). It is not recommended to machine wash the device more than 15 times (average: one wash per month). Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess

water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2042801 (2022-10)
THUASNE UK Ltd**
Thuasne SAS - SIREN/RCS Nanterre 542 091 186