**THUASNE**

# Ligastrap® Malleo

<b>fr</b>	Chevillère élastique proprioceptive avec sangles stabilisatrices	5
<b>en</b>	Proprioceptive elastic ankle brace with stabilising straps	6
<b>de</b>	Elastische propriozeptive Knöchelbandage mit Stabilisierungsbändern	7
<b>nl</b>	Proprioceptieve elastische enkelbrace met stabiliserende banden	9
<b>it</b>	Cavigliera elastica proprioceppiva con cinghie stabilizzatrici	10
<b>es</b>	Tobillera elástica propioceptiva con cinchas estabilizadoras	12
<b>pt</b>	Tornozeleira elástica proprioceptiva com fitas estabilizadoras	13
<b>da</b>	Proprioceptisk ankelbind med stabiliserende stropper	15
<b>fi</b>	Proprioseptiivinen joustava nilkkatuki, jossa stabiloivat hihnat	16
<b>sv</b>	Proprioceptiv elastisk ankelortos med stabiliserande remmar	17
<b>el</b>	Ελαστική ιδιοδεκτική επιστραγαλίδα με ιμάντες σταθεροποίησης	19
<b>cs</b>	Elastická proprioceptivní kotníková ortéza se stabilizačními pásky	20
<b>pl</b>	Elastyczna proprioceptywna orteza skokowo-stopowa z taśmami stabilizującymi	21
<b>lv</b>	Proprioceptīvs elastīgs potītes pārsējs ar stabilizēšanas siksnām	23
<b>lt</b>	Tamprus propriocepriją gerinantis čiurnos įtvaras su stabilizuojančiais diržais	24
<b>et</b>	Proprioseptiivne elastne hüppeliigesetugi stabiliseerivate rihmadega	26
<b>sl</b>	Elastična proprioceptivna opornica s trakovi za stabiliziranje	27
<b>sk</b>	Proprioceptívna elastická členková ortéza so stabilizačnými páskami	28
<b>hu</b>	Proprioceptív rugalmas bokarögzítő stabilizáló pántokkal	30
<b>bg</b>	Проприоцептивна еластична шина за глезен със стабилизиращи каишки	31
<b>ro</b>	Glezneră elastică proprioceptivă cu fașe stabilizatoare	33
<b>ru</b>	Эластичный проприоцептивный ортез на лодыжку со стабилизирующими ремнями	34
<b>hr</b>	Elastični proprioceptivni steznik za gležanj s remenima za stabilizaciju	36
<b>zh</b>	帶穩定搭扣的彈力本体感護踝	37
<b>ar</b>	دعامة كاحل مرنة متعلقة باستقبال الحس العميق مع أحزمة للتثبيت	39

	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

<b>fr</b>	Tour de cheville
<b>en</b>	Ankle circumference
<b>de</b>	Knöchelumfang
<b>nl</b>	Omvang van de enkel
<b>it</b>	Giro caviglia
<b>es</b>	Circunferencia del tobillo
<b>pt</b>	Perímetro do tornozelo
<b>da</b>	Anklens omkreds
<b>fi</b>	Nilkanympärys
<b>sv</b>	Runt fotleden
<b>el</b>	Περίμετρος αστραγάλου
<b>cs</b>	Obvod kotníku
<b>pl</b>	Obwód kostki

<b>lv</b>	Potītes apkārtmērs
<b>lt</b>	Kulkšnies apimtis
<b>et</b>	Pahkluu ümbermõõt
<b>sl</b>	Obseg gležnja
<b>sk</b>	Obvod členka
<b>hu</b>	Boka kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința gleznei
<b>ru</b>	Окружность голени
<b>hr</b>	Opseg gležnja
<b>zh</b>	腳踝尺寸

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---

## en PROPRIOCEPTIVE ELASTIC ANKLE BRACE WITH STABILISING STRAPS

### Description/Destination

This device is a proprioceptive elastic ankle brace providing joint support. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Polyamide - Elastane - Polyester - Elastodiene - Polyurethane.

### Properties/Mode of action

Elastic compressive knitting (class 3 - 20-36 mmHg / 271-48 hPa).

Double lateral support strapping to maintain flexion-extension.

Removable lateral support strap.

Tabs to make it easier to pull on.

### Indications

Mild ankle sprain (acute phase).

Post-operative, post-traumatic residual pain and/or edema.

Joint instability/laxity.

Proprioceptive support for rehabilitation.

Sport activities resumption.

Compression for scar/burns healing.

### Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

### Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application. Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient. Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

### Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Slip on the ankle brace. The top comprises the closure tab.

Close this tab (with the self-adhesive strip) to obtain a snug fit.

Pull the first strap, wrap it around the ankle and fix it on the felt pad located at the base (figures ❶ and ❷).

Repeat this operation with the second strap (figure ❸) : the ankle must be firmly supported laterally with unhindered flexion-extension.

If necessary, the ankle support may be reinforced. To do this, lay out the removable strap on the ground, with the self-adhesive parts upwards.

Then place the foot on its midpart (figure ❹), and pull the two ends up.

Make an "8" crossover above the foot and fix the ends of the strap on the felt pad at the top of the ankle brace (figure ❺).

For comfort during wear make sure the straps are not creased.

### Care/Maintenance

Product can be washed in accordance with the conditions shown on this notice and on the label. Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

### Storage

Store at room temperature, preferably in the original packaging.

### Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2023401 (2021-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros