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**en** This barrier mask is not a replacement for application of protective measures (regular hand-washing, social distancing, reduction of contact with other people). It adds a physical barrier enabling the user to protect his/her environment from the projection of particles. It can also provide the user with limited protection against infectious agents.

**It is essential to put your mask on properly to ensure good protection.**

Wash your hands with soap and water or alcohol based sanitizer before putting on a mask.

**Make sure the mask is clean and dry before you put it on.**

- Unfold the mask ❶. The OUTSIDE is identified by the Thuasne logo.
- Place the elastic straps behind your ears. ❷
- Adjust the nose piece at the top of the mask. ❸
- Make sure the mask is properly in position by checking the seal. To do this, cover the mask with a plastic film; when you breathe in, the mask should flatten against your face ❹. Check that there is no air blown into the eyes when you breathe out hard.
- Once the mask is in place, breathe normally.

Always check that the mask is properly adjusted and that it covers your mouth and nose. It is recommended that this mask be worn over bare skin; beards may reduce the filtration efficacy outside the target range.

The mask can be worn for 4 hours continuously between each wash.

When wearing a mask, avoid touching it and do not move it.

Wash your hands with soap and water or alcohol-based sanitizer each time you touch a used mask.

If you need to eat or drink, change your mask.

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**fr**

This mask is neither a medical device within the meaning of Regulation (EU) 2017/745 (surgical masks), nor an item of personal protective equipment within the meaning of Regulation (EU) 2016/425 (such as an FFP2-type filtering mask).

## Name

AFNOR SPEC S76-001 barrier mask

This mask is for non-medical use (NMF) only and is aimed at personnel working in jobs or tasks involving regular contact with the public. It protects against the emission or dispersal of droplets of saliva or upper airway secretions that could contain transmissible infectious agents.

## Precaution

To protect your health and that of others, it is very important to comply with the product information leaflet. If you are ill, this mask is not suitable; consult your doctor for advice. This mask is not intended for healthcare professionals.

## Washing instructions

**Your Thuasne barrier mask is reusable and must be washed with care (according to ANSM notification of 3/25/2020 revised 11/2/2020) :**

- Wash the mask at 40°C on a complete wash cycle lasting at least 30 minutes.
- Wash the mask every day, after each use.
- Tumble-drying and ironing recommended.
- Store at room temperature, in original packaging.

**This mask can be washed 50 times\* :**

Masks must be completely dry before use.

Mask disposal procedure :

- using the specific procedure for the disposal of infectious clinical waste.
- in a carefully sealed plastic bag if disposal via the infectious clinical waste procedure is not possible.

A washing table can be downloaded at :

<https://fr.thuasne.com/fr/masque>

**Composition:**

Polyamide, polyester, contains latex.

Nose piece: steel/polypropylene.

**Size:** adult.

**Contraindications**

- Known allergy to any of the components.
- Do not place the product in direct contact with broken skin.
- Severe respiratory insufficiency.

**Type of product :**

General public mask, filtration greater than 90%, in accordance with the information note of March 29, 2020, revised January 28, 2021 :

- filtration efficacy for 3 µm particles emitted > 90%
- air permeability for a pressure drop of 100 Pa > 96 L.m<sup>2</sup>.s<sup>-1</sup>

\* IFTH tests ref. 20-01731

The material complex has been tested and complies with the technical characteristics of masks for general public use for up to **100 washes** performed in the laboratory, without taking into account real use conditions.