THUASNE

Pedipro


fr	Talonnets silicone	5
en	Silicone heelpieces	6
de	Ferseneinlagen aus Silikon	8
nl	Siliconen zolen	10
it	Alzaticchi in silicone	12
es	Talonerías de silicona	14
pt	Palmilhas de calcanhar de silicone	15
da	Indlægshæle i silikone	17
fi	Silikonikantapäätet	19
sv	Silikoninlägg	21
el	Υποπτέρνια σιλικόνης	22
cs	Silikonové podpatěky	24
pl	Podpiętki silikonowe	26
lv	Silikona papēži	28
lt	Silikoninės pakulnės	30
et	Silikonist kannapadjad	31
sl	Silikonski petni nastavki	33
sk	Silikonové podpätenky	35
hu	Szilikon sarokemelő	36
bg	Силиконови подложки за пета	38
ro	Talonete din silicon	40
ru	Силиконовые подпяточники	42
hr	Silikonski jastučići za petu	44
zh	硅胶足跟护垫	46
ar	وسائد كعب سيليكونية	49

		Europe	UK
	1	36 - 39	3 ^{1/2} - 5 ^{1/2}
	2	40 - 46	6 ^{1/2} - 11

fr	Pointure - Europe - UK
en	Shoe size - Europe - UK
de	Schuhgröße - Europa - UK
nl	Schoenmaat - Europa - VK
it	Numero di scarpa - Europa - Regno Unito
es	Número de pie - Europa - Reino Unido
pt	Número do sapato - Europa - Reino Unido
da	Størrelse - Europa - UK
fi	Kengän koko - Eurooppa - Yhdistynyt kuningaskunta
sv	Skostorlek - Europa - Storbritannien
el	Νούμερο υποδημάτων - Ευρώπη - ΗΒ
cs	Velikost obuvi - Evropa - Spojené království
pl	Numer buta - Europa - Wielka Brytania
lv	Apavu izmērs - Eiropa - Apvienotā Karaliste
lt	Avalynės dydis - Europa - JK
et	Suurus - Euroopa - Ühendkuningriigid
sl	Velikost obutve - Evropa - UK
sk	Veľkosť - Európa - Spojené kráľovstvo
hu	Cipőméret - Európa - UK (Egyesült Királyság)
bg	Размер - Европа - Обединено кралство
ro	Mărimea pantofului - Europa - Regatul Unit
ru	Размер обуви - Европа - Великобритания
hr	Broj cipela - Europa - UK
zh	鞋码-欧洲-英国
ar	المقاس - أوروبا - المملكة المتحدة

Décharge	Restaure l'équilibre	Correction biomécanique
Off-loading	Balance restoration	Biomechanical correction
Entlastung	Stellt das Gleichgewicht wieder her	Biomechanische Korrektur
Losmaken	Herstelt het evenwicht	Biomechanische correctie
Scarico	Restauro l'equilibrio	Correzione biomeccanica
Descarga	Restauro el equilibrio	Corrección biomecánica
Alívio do peso	Restauro o equilibrio	Correção biomecânica
Aflastning	Genopretter ligevægten	Biomekanisk korrektion
Vapautus	Palauttaa tasapainon	Biomekaaninen korjaus
Avlastning	Förbättrar balansen	Biomekanisk korrigering
Αποφόρτιση	Αποκαθιστά την ισορροπία	Βιο-μηχανικός έλεγχος
Snižení zátěže	Obnovuje rovnováhu	Biomechanická korekce
Odciążenie	Produkt przywraca równowagę	Korekcja biomechaniczna
Atbrīvošana	Līdzsvara atjaunošana	Biomehāniskā korekcija
Apkrovos sumažinimas	Pusiausvyros palaikymas	Mechaninė biologinė korekcija
Koormuse vähendamine	Taastab tasakaalutunnetuse	Biomehaaniline korrektsioon
Razbremenitev	Vzpostavi nazaj ravnovesje	Popravek biomehanske motnje
Uvoľnenie	Obnovenie rovnováhy	Biomechanická korekcia
Tehermentesítés	Az egyensúly helyreállításá	Biomechanikus korrekció
Отбремяванe	Възстановява баланса	Биомеханична корекция
Descărcare	Restabilirea echilibrului	Corecție biomecanică
Разгрузка	Восстановление равновесия	Биомеханическая коррекция
Rasterečenje	Ustovavljanje ravnoteže	Biomehānička korekcija
减压	恢复平衡	生物力学矫正
تخفيف ثقل	إعادة التوازن.	تصحيح ميكانيكي حيوي

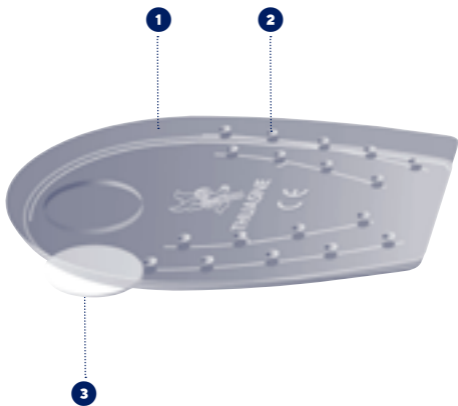fr

TALONNETTES SILICONE

Description/Destination

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en SILICONE HEELPIECES

Description/Destination

Silicon provides even distribution of pressure and shock absorption. ❶

Ventilation channels allow air flow in the shoe. ❷

Unique self-adhesive system helps them to stay in place in the shoe. ❸

Composition

Silicone - polyamide.

Properties/Mode of action

Off-loading.

Balance restoration.

Biomechanical correction.

Indications

Heel and foot pain (metatarsalgia, plantar fasciitis).

Rheumatoid foot.

Relief for tendinitis.

Contraindications

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

Precautions

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional. Choose the appropriate size to fit, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

Undesirable side-effects

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Position the self-adhesive tab under the silicone surface of the heel insert (self-adhesive surface against the heel insert).

Clean the interior of the shoe.

Remove the protective paper from the adhesive tab.

Place the heelpiece flat in the shoe, pressing the rim of the product against the heel of the shoe.

Press down hard with your hand to make sure the tab is properly stuck to the shoe.

Repeat this operation with the other heelpiece.

THUASNE has supplied you with extra tabs so that you can remove the heel inserts and use in different shoes.

Care

It is recommended to wash the product every day, or after each use. Hand wash. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking

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©Thuasne - 2030702 (2022-07)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186